



ANTECH IMAGING SERVICES

<https://info.antechimagingervices.com>

Saving Dental Radiographs as a study (Progeny)

1. Make sure all images from the procedure are on main screen.
2. Click "**Studies**" tab in left column.
 - a. "Name" - This is the Study Name, and we suggest entering the date of procedure.
 - b. "Description" - Use your Dr. initials or other description.
3. Click "**Save.**"
 - a. This will make a study in the column.
4. Select the study you want to send and click "**Publish.**"
5. When you click "**Publish**", it will download the images to the server or wherever Progeny has been instructed to have the database during configuration/set-up.
 - a. See pages 14-17 of your Progeny Imaging Installation Guide for PACS configuration instructions.
 - b. A copy of the Installation Guide is available on our website [here](#).
 - c. Contact our technical support team if you need further assistance with configuration.